VERMEIREN

4-Light

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

Instructions to the specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: A, February 2012

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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4-wheels rollator 4-light

1. Technical details

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

| | | | |
|--|---------------------------------------|-------------|---------|
| Make | Vermeiren | | |
| Address | Vermeirenplein 1/15, B-2920 Kalmthout | | |
| Type | Rollator | | |
| Model | 4-light | | |
| Width | 550 mm | | |
| Length (unfolded) | 750 mm | | |
| Maximum height | 1015 mm | | |
| Minimum height | 795 mm | | |
| Seatheight | 560 mm | | |
| Seatwidth | 400 mm | | |
| Seatdepth | 220 mm | | |
| Weight | 8 kg | with basket | 8,45 kg |
| Wheels | 8 x 1 ¼ (Ø 200 mm) | | |
| Brakes | Multifunctional brake | | |
| Max. inclination forwards | 15° | | |
| Max. inclination backwards and sideways | 10° | | |
| Maximum user weight | max. 130 kg | | |
| Maximum basket weight | max. 5 kg | | |
| Turning width | 1760 mm | | |
| Storage temperature | +5°C - +41°C | | |
| We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg / ° | | | |


First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

Before using the rollator, read the instruction manual carefully: it will familiarise you with this product.

You will also be given hints about the care and service of your rollator. Please remember that observance of our hints would keep your rollator in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, please consult your specialist dealer.
To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

2. Components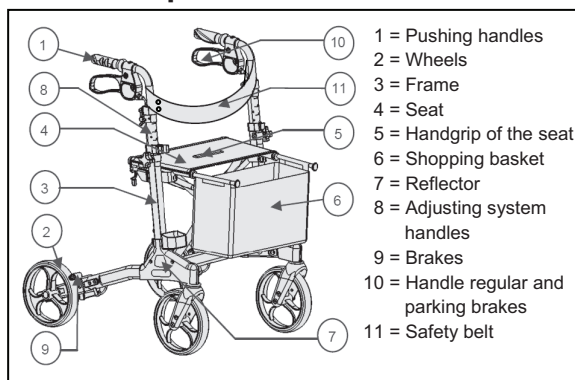

3. Explanation of symbols

- Maximum mass
- Indoor and outdoor use
- CE conformity
- Read the instruction manual before use
- Observe the safety instructions

4. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- rollator frame with seat, 4 wheels (mounted)
- height adjustable pushing handles with brakes
- shopping basket
- safety belt
- manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

5. Intended use

⚠ WARNING: Risk of falling - Do not use the rollator if you have weak arm function or poor equilibrium.

The rollator will compensate the lack of adequate support by the legs by leaning on, or moving with, the rollator with usage of the upper limbs or chest. The rollator will provide a better stability, support and safety during walking.

However, it cannot be used by persons with a visible weakened hand and/or arm function and also not by persons with a balance problem.

The rollator can be used indoor and outdoor. The rollator should only be used on surfaces where all four wheels are touching the ground. Not to be used on rocky terrain or steep hills (see technical details: max. inclination) so the rollator can become unstable.

The product is not meant for transport of people and/or goods. The seat can only be used when the parking brake is locked.

6. Assembly (Fig. 1)

⚠ CAUTION: Chance of pinching - Do not place fingers between the components of the rollator.


⚠ CAUTION: Risk of clamping - Keep fingers away from moving parts of the rollator.

1. Unfolding rollator

- Remove the attachment plate ① (right side of frame) from the left tube of the seat ②.
- Unfold the rollator by pushing on the handles.
- Push on the tubes of the seat ② to unfold the rollator further.
- Click the tubes of the seat ② in the black tube holder ③.
- Mount the attachment plate ① to the pen ④.

2. Adjust the pushing handles to the desired height (see further).

Fig. 1 – Folding / Unfolding rollator

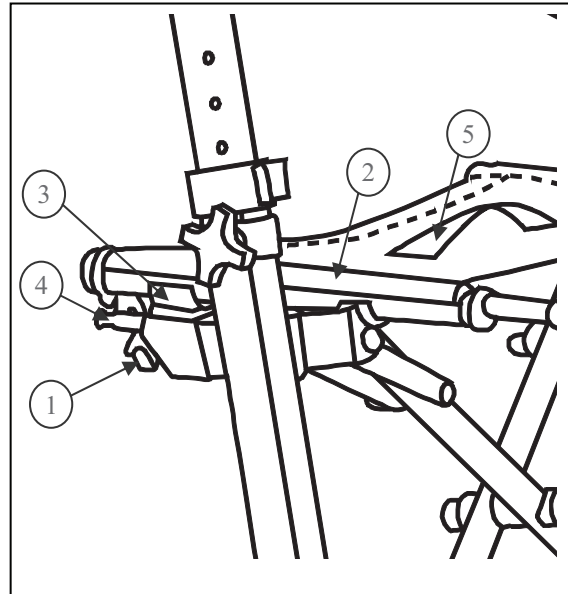7. Dismantling (Fig. 1)

⚠ CAUTION: Chance of pinching - Do not place fingers between the components of the rollator.

⚠ CAUTION: Risk of clamping - Keep fingers away from moving parts of the rollator.

1. Adjust the pushing handles to the minimum position (see further).

2. Folding rollator


- Remove the attachment plate ① from the pen ④.
- Pull lightly on the handgrip of the seat ⑤ till the tubes ② comes out the black tube holder ③.
- Fold the rollator by pushing on the handles or pulling on the tubes of the seat ②.
- Click the attachment plate ① to the tube of the seat ②.

8. Pushing handles (Fig. 2)

- ⚠ **CAUTION: Risk of falling - Firmly fix the handles before use of the rollator.**
- ⚠ **CAUTION: Risk of falling - Do not exceed the maximum adjustment, your rollator will not be stable anymore.**

Fig. 2 – Adjustment pushing handle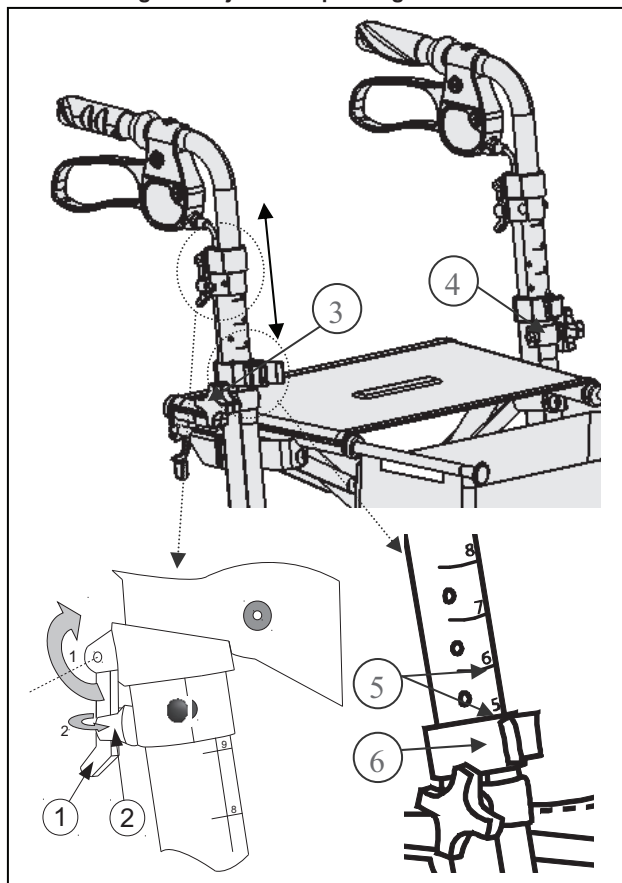

The pushing handles can be adjusted in height by following steps:

- Pull the lever ① up.
- Turn the button (just below the lever ①) ② to the outside of the frame.
- Remove the star knob ③ from the screw.
- Turn the clip ④ around the axle of the pushing handle.
- Remove clip ④.
- Adjust the handles to the desired position. When standing straight, the handles should be as high as your wrists. Leave your arms hanging loose. The minimum and maximum adjustments can be seen on the marks on the inner tubes which slide into the frame. **Check that the mark line ⑤ is at the same position with the outer tube ⑥ where the pushing handle slides into the frame. Otherwise you could damage the mechanism.**
- Mount clip ④ in the hole of the frame.


- Turn the clip ④ around the axle of the pushing handle until the hole comes over the screw.
- Mount the star knob ③ to the screw.
- Turn the button ② to the begin position (to the inside).
- Push the lever ① down, till the lever ① is mounted in button ②.

9. Brake system (Fig. 3)

- ⚠ **WARNING: Good operation of the brakes is influenced by wear and contamination of the wheels (water, oil, mud, ...) - Check the condition of the wheels before each use.**
- ⚠ **WARNING: The brakes can wear - Check the operation of the brakes before each use.**

The rollator has been equipped with a multifunctional brake system. The brake handles underneath the pushing handles are regular brakes as well as parking brakes. For the usage of a short brake (regular brakes), pull the brake handle upwards (toward the pushing handle). For the usage of a long brake (parking brake), push the brake handle downwards till you hear a locking sound of the brake. The wheels (use both parking brakes left and right simultaneously) are now blocked until you pull the parking brakes upwards again.

Fig. 3 – Pushing handle / brake

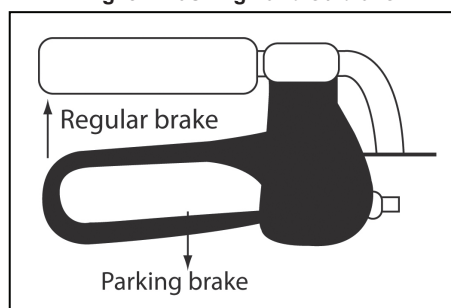10. Basket

- ⚠ **WARNING: Risk of falling, tipping over - Do not overload the shopping basket.**

The shopping basket is meant to be mounted on the front side. Mount the shopping basket with the four clips to the two tubes.

The maximum carrying weight from the shopping basket is 5 kg.

11. Safety instructions

- Ensure there are no stones or any other objects blocking the regular usage of the wheels of the rollator.
- Ensure the brakes or the rollator are always in parking-mode before using your rollator. When you are standing comfortable and stable, you may unlock the parking brakes.
- Risk of burns – Be careful when driving in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching.
- Before every usage, check if the locking handle for your pushing handles is firmly locked.
- Verify if the safety against unwanted folding/unfolding is functioning properly.
- Only proceed moving with your rollator when you are standing comfortably and when both your hands are resting on the pushing handles.
- Do not hold the pushing handles with wet hands. If you do so, you can lose your grip and lose your balance.
- When you detect that your body is getting tired, lock the parking brakes of your rollator and have a seat on the bench.
- **The maximum users weight for your rollator is 130 kg** and can only be used on a flat underground. Do not use your rollator on a staircase or bumps higher than 40 mm.
- When using the rollator outdoors, find yourself flat and stable areas to use your rollator. Avoid curbs and/or deep holes. Find a place where the curbs are bevelled.
- Do not proceed with your rollator into traffic. Always stay on the pavement.
- Avoid bumpy or sandy roads. Otherwise you might lose control over the stability of your rollator.
- Avoid roads with a very steep inclination.
- Do not use your rollator to transport people or items.

- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

12. Care

It is recommended to use a soapy solution for cleaning. The cleaning liquid should in any case have a pH value of at least 6. Do not use a cleaning agent which contains solvents. Observe the instructions of the cleaning liquid used. Use a damp cloth and lukewarm water for cleaning. Avoid soaking. Do not use metal brushes or any other sharp-edged or pointed cleaning utensils. In the case of damage to the varnish or the woodwork, consult your specialist dealer. Regular care would ensure that your rollator is preserved in a perfectly functional condition.

13. Disinfection

For sterilisation use only disinfectants suitable for treating varnished metal and wood surfaces. Observe the instructions of the cleaning liquid used. Use only proved processes (for scrubbing disinfection) and disinfectants included in the list of the Robert Koch Institute (information can be obtained from: www.rki.de). Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

14. Repairs / Service / Inspection

Lifetime of the rollator is influenced by its use, storage, regular maintenance, servicing and cleaning.

You should use only genuine VERMEIREN replacement parts. Repairs and renovations may only be undertaken by trained persons. Make use of the services offered by your specialist dealer. He would gladly assist you further in regard to service and repairs.

Before you resume use of the rollator, it should be serviced according to the following criteria:

- Completeness
- Foldingmechanism
- Check the castors (stability, tracking, fixing screws)
- Frames (deformation, stability, connections)
- Pushing handle (safe locking from height adjustment system, good support from the handles)
- Brakes
- Seat, safety belt and shopping bag on wear or damage
- Cleaning

- **Damaged varnish (remove possible rust and repaint)**

Every 8 weeks:

- Lubricating (moving parts shall be greased)

Every 6 months or for every new user, it should be serviced according to the following criteria:

- General review
- Disinfection

15. Shipping and storage

The shipping and storage of the rollator shall be according following instructions:

- Store in a dry place (between +5°C and +41°C).
- The relative humidity of the air should be between 30% and 70%.
- Provide sufficient covering or packaging to protect the rollator from rust and foreign bodies (e.g. salt water, sea air, sand, dust).
- The rollator must be stored without being subjected to strains. (Do not put too heavy parts on the rollator, not clamping between something, ...)

16. Warranty

Excerpt from the "General Business Conditions".

(...)

5. The guarantee period for warranty claims is 24 months.

(...)

The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts.

Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

17. Disposal

When disposing of the rollator, contact your local disposal centre or return the product to your specialist dealer who, after submitting it to a hygienic procedure, will be able to send it back to the manufacturer who will dispose of and recycle it correctly, separating it into its component materials.

Packaging materials can be taken to disposal or recycling centers or to your specialist dealer.

18. Declaration of conformity

The manufacturer or his authorized representative :

N.V. VERMEIREN N.V

Address :

Vermeirenplein 1/15
2920 Kalmthout
Belgium

declares under his sole responsibility that the CE marked medical devices :

| | |
|---------------|-----------|
| Productgroup: | Rollators |
| Brand: | Vermeiren |
| Type: | 4-light |

have been classified as class I, according to annex IX MDD 93/42/EEC, rule 1,

and is manufactured in full conformity with the European instructions below - including the latest modifications - and with the national law, that organizes this directions :

Medical devices directive MDD 93/42/EEC

and is in conformity with the relevant European harmonized standards:

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ENGLISH

CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim this warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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N.V. VERMEIREN N.V.
Vermeirenplein 1/15
B-2920 Kalmthout
BELGIUM

SERVICE

The rollator was serviced:
Le rollator roulant contrôlé:
De rollator is gecontroleerd:

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Notes

A series of horizontal dotted lines for writing notes.

Belgium

N.V. Vermeiren N.V.

Vermeirenplein 1 / 15
B-2920 Kalmthout
Tel: +32(0)3 620 20 20
Fax: +32(0)3 666 48 94
website: www.vermeiren.be
e-mail: info@vermeiren.be

France

Vermeiren France S.A.

Z. I., 5, Rue d'Ennevelin
F-59710 Avelin
Tel: +33(0)3 28 55 07 98
Fax: +33(0)3 20 90 28 89
website: www.vermeiren.fr
e-mail: info@vermeiren.fr

Italy

Reatime S.R.L.

Viale delle Industrie 5
I-20020 Arese MI
Tel: +39 02 99 77 07
Fax: +39 02 93 58 56 17
website: www.reatime.it
e-mail: info@reatime.it

Poland

Vermeiren Polska Sp. z o.o

ul. Łączna 1
PL-55-100 Trzebnica
Tel: +48(0)71 387 42 00
Fax: +48(0)71 387 05 74
website: www.vermeiren.pl
e-mail: info@vermeiren.pl

Spain

Vermeiren Iberica, S.L.

Trens Petits, 6. - Pol. Ind. Mas Xirgu.
17005 Girona
Tel: +34 902 48 72 72
Fax: +34 972 40 50 54
website: www.vermeiren.es
e-mail: info@vermeiren.es

Germany

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Wahlerstraße 12 a
D-40472 Düsseldorf
Tel: +49(0)211 94 27 90
Fax: +49(0)211 65 36 00
website: www.vermeiren.de
e-mail: info@vermeiren.de

Austria

L. Vermeiren Ges. mbH

Winetzhammerstraße 10
A-4030 Linz
Tel: +43(0)732 37 13 66
Fax: +43(0)732 37 13 69
website: www.vermeiren.at
e-mail: info@vermeiren.at

Switzerland

Vermeiren Suisse S.A.

Hühnerhubelstraße 59
CH-3123 Belp
Tel: +41(0)31 818 40 95
Fax: +41(0)31 818 40 98
website: www.vermeiren.ch
e-mail: info@vermeiren.ch

The Netherlands

Vermeiren Nederland B.V.

Domstraat 50
NL-3864 PR Nijkerkerveen
Tel: +31(0)33 2536424
Fax: +31(0)33 2536517
website: www.vermeiren.com
e-mail: info@vermeiren.be

Czech Republic

Vermeiren ČR S.R.O.

Sezemická 2757/2 - VGP Park
193 00 Praha 9 - Horní Počernice
Tel: +420 731 653 639
Fax: +420 596 121 976
website: www.vermeiren.cz
e-mail: info@vermeiren.cz