




# CGM Neonatal Crib



Chang Gung Medical Technology Co., Ltd.

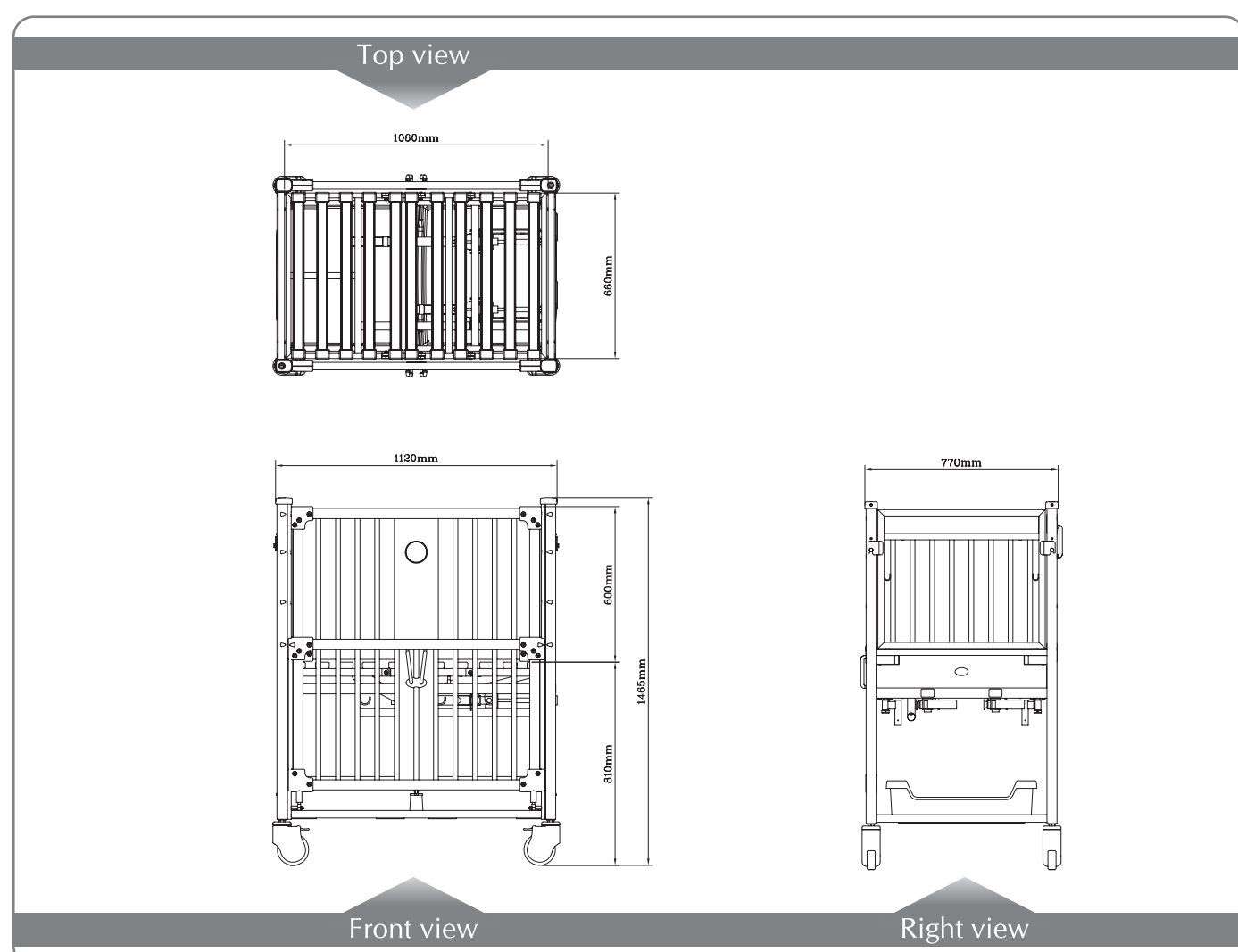

## Specification

Dimension (I)		Standard Accessories	
● Overall Size	L 112 x W 77 cm	● Fixed I.V. Pole	1PC
● Sleeping Surface	L 106 x W 66 cm	Optional Accessories	
● Siderail	L 102 x H 60 cm	● Mattress	
Dimension (II)		● Oxygen Tank Holder	
● Overall Size	L 159 x W 77 cm	● I.V. Pole	
● Sleeping Surface	L 153 x W 66 cm		
● Siderail	L 149 x H 60 cm		
Positioning			
● Backrest	0 - 75 °		
● Knee Gatch	0 - 40 °		

CGM reserves the right to make changes without notice in design and specification.



**Chang Gung Medical Technology Co., Ltd.**

Headquarter: 11F., No.201-4, Dun-Hwa North Rd., Taipei 105, Taiwan, R.O.C.  
Tel: 886-2-87122948 Fax: 886-2-25140620

E-mail: cgmc@cgmc.com.tw http://www.cgmc.com.tw



CGEAN1012-1



## Neonatal Crib (Model: CKP / CKQ)



### 1 Fixed I.V. Pole



- The stainless I.V. Pole can be placed at the head and the end of bed for hanging I.V. bottle and IVAC, according to medial care requirement.
- The height of the I.V. Pole is adjustable from 108 to 148 cm, measured from bed surface.

### 2 Caster

5" casters with dust-proof cover can prevent effectively dust from rolling inside the bearing.

### 3 Siderail Elevation Grip



- The smart grip is only for adults to move up/down and lock/unlock the siderail. By functional grips with 4-fixed position on both sides of tracks, the whole security system has been greatly upgraded.
- Gas-spring mechanism set on both sides of the siderails is to assist the siderail up and down easily.

### 4 Storage Bin

Two plastic bins are under the base of bed for storages of towels, diapers, clothes and toys.

Lovely appearance with vivid color

Provide the infant comfortable environment



### 5 Hi-Lo Crank



The Backrest and Knee Gatch cranks are located at the foot-end of beds, and can be completely stored.

### 6 Active I.V. Pole Socket (Optional)

I.V. Pole sockets are hidden under the foot-end frame and can be rotated out to use, and the I.V. Pole can be stored below the bed frame.

### 7 Siderail

- The fence-style siderail is made of Aluminum without rust problem.
- The champagne color is lovely and popular.
- The safety height is conformed to international standard.

### 8 Siderail ABS Cover

The ABS-made cover can keep the siderail from damages and increase its practicability. It also features warm color, easy-clean, smooth and comfortable touch.

### 9 Platform

Plastic bar platform provides excellent ventilation and is easy to clean.

### 10 Removable Head & Foot Board (Optional)



The Head & Foot board can be quickly removed for first-aid while emergency.

